

VEUOGH

GENETIC TESTS

 **IMPORTANT**

IF YOU HAVE DIFFERENT TYPES OF SAMPLE TO TAKE, THE BLOOD SAMPLE SHOULD ALWAYS BE TAKEN LAST.

EQUIPMENT

To perform this sampling, we provide you with :

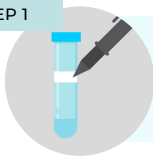
• 1 swab.

• 1 tube with blue cap.

 **IMPORTANT****The tube may not be rinsed.**

It contains guanidinium thiocyanate which is essential for the analysis.

- Do not eat, drink, smoke or chew gum 30 minutes before taking the saliva sample.

STEP 1

Write on the label of the provided cup: your name, first name and date of birth.

STEP 2

Wash your hands thoroughly.

STEP 3

Open the sterile package and take out the swab. **Be careful not to touch the end of the swab with your fingers.**

STEP 4

Insert the swab into your mouth and **rub the inside of your cheeks for 1 to 2 minutes.**

STEP 5

Transfer the swab to the tube and **break the swab shaft at the red line.**

STEP 6

Close the tube tightly and place it in the Emball'Infor bag.